

SneakPeek Clinical

A >99% accurate predictive fetal sex test for use starting six weeks into pregnancy



What is SneakPeek?

SneakPeek is a blood test that helps your patients learn their baby's sex as soon as **six weeks** into pregnancy with **>99% accuracy**.¹

- Sex results only, no disease screening
- Self-pay pricing, non-insurance based
- Fast turnaround time, results as soon as next-day



Trusted by more than
750,000 parents



Why offer SneakPeek to your patients?

SneakPeek offers you the opportunity to meet your patient's needs at a stage in their pregnancy journey where they want to find out their baby's sex as early as possible. For some patients, having this information has allowed them to bond with their baby sooner, plan ahead, and connect with family and friends.

With that question answered, healthcare providers can shift the focus to the baby's and the pregnancy's health.

SneakPeek correctly identified fetal sex in 113 of 113 pregnancies at 6 weeks gestation¹

Number of Samples	113
Gestational Age of Participants	6 weeks
Accuracy at 6 weeks	113/113 (100%)
Sensitivity at 6 weeks (Male Determination)	48/48 (100%)
Specificity at 6 weeks (Female Determination)	65/65 (100%)

1. Szucs HD, Asprer L, Abunadi N, Milot H, Cacia S, Jacob C (2022) SneakPeek Early Gender Test: The Earliest and Most Accurate Method for Fetal Sex Determination at 6-Weeks Gestation. Clinics Mother Child Health. 19:419.

The SneakPeek® Science

SneakPeek uses a blood draw to gather the cell-free fetal DNA (cffDNA) that circulates in the pregnant person's blood. From a small sample collected at your office and sent to SneakPeek Labs, the cffDNA is used to detect Y chromosomes. If Y chromosomes are found, then the predicted sex of the baby is male. Otherwise, the baby is predicted to be female.

How does it work for your clinic?

1. Kits are mailed to have on hand at your facility (free of charge).
 - Included: Kits, postage, advertising materials, blood collection solution (if needed)
2. Blood is collected onsite at your facility. You collect payment from your patient at this time.
3. Sample is shipped to SneakPeek Labs overnight.
4. Within 4-8 hours of sample arrival, results are emailed directly to you and your patient.
5. SneakPeek invoices you after results are reported.

Benefits of becoming a SneakPeek provider

- Optional placement on the SneakPeek Clinical test store locator (+6M visitors/year) to introduce new patients to your business
- Complimentary marketing materials
- Simple and minimal work for your business



Contact Us

<https://sneakpeektest.com/contact-us/>

Learn More

<https://sneakpeektest.com/become-a-sneakpeek-clinical-provider/>

**Join over 1,850 clinics offering
SneakPeek Clinical!**



Scan to learn more